



Instructions


Step 1: Fill out Patient Information, and circle appropriate Laterality and Digit.

Step 2: Place Affected Hand and Digit on Grid as shown below.

Step 3: Place a dot on either side of your affected finger just beyond the webspace of your hand. This will be where your splint ends. See the position of the **Light Blue Dots** Below.

Step 4: Place a dot on either side of your affected finger at the level of your close knuckle just along the middle skin crease of your knuckle. See the position of the **Green Dot** on the Finger Diagram Below.

Step 5: Place a dot on either side of your affected finger at the level of your far knuckle along the skin crease closest to you. This is how long your splint will be. See the Position of the **Dark Blue Dot** on the Finger Diagram Below.

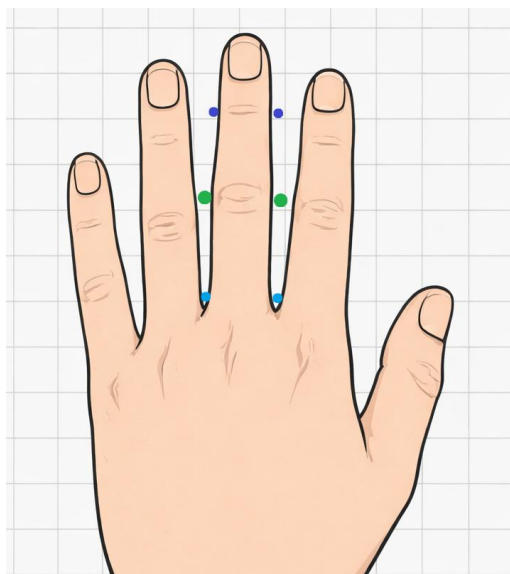

Patient Name

Laterality

Right

Left

Digit

Index

Middle

Ring

Small



Please Take a Picture of the completed Grid Above ensuring all 4 corners are visible and email to adept.ortho@gmail.com along with any patient specific information required for your treatment plan.